

Negative Pressure Wound Therapy

To turn unit On:

- Press the **ON** or **OK** button.

To reach 'option mode' to input settings:

- Hold the up arrow button and the down arrow button simultaneously.
 - Options for choices **Continuous, Intermittent,** or **Language** will appear on screen.

To input Continuous settings and start therapy:

- Press **OK** while highlighted over **Continuous**.
- The appearing screen is to change setting by pressing the up or down buttons.
 - Pressure will increase or decrease by -5mmhg increments.
 - Press **OK** when desired pressure is reached.
- The screen remains yellow/idle and allows for the pressure to be reviewed and confirmed.
- Press the **OK** button to START therapy, screen will turn green.
 - Pressure will slowly pull down by -5mmhg increments until desired level reached.
 - The large number present on the screen is the therapy level, the small number to the right is the **input** pressure level.

- A small bar will appear on the left hand side of the screen, it reads **too tight** at the top and **not tight** at the bottom. The bar will raise to **too tight** if not enough air flow is within the system; **not tight** will cause an alarm if there is a large air leak.
- The screen will automatically lock after 15 minutes of uninterrupted therapy; however, if clinician prefers to expedite the lock, press and hold the up arrow and down arrow buttons simultaneously, a small padlock will appear in the lower left hand corner.

To stop or pause therapy from Continuous mode:

- Unlock unit if a padlock to the lower left hand side near the battery is present.
 - To unlock, press and hold the up and down arrow buttons simultaneously until the padlock disappears from the LED screen.
- To pause, press the **On** or **OK** button once.
- To turn unit off, press and hold the **Off** or **C** button for 3 seconds.
- The unit is powered off when the LED screen is dark.

